Pionier der multidirektionalen Winkelstabilität
Pioneer of multidirectional locking screw technology

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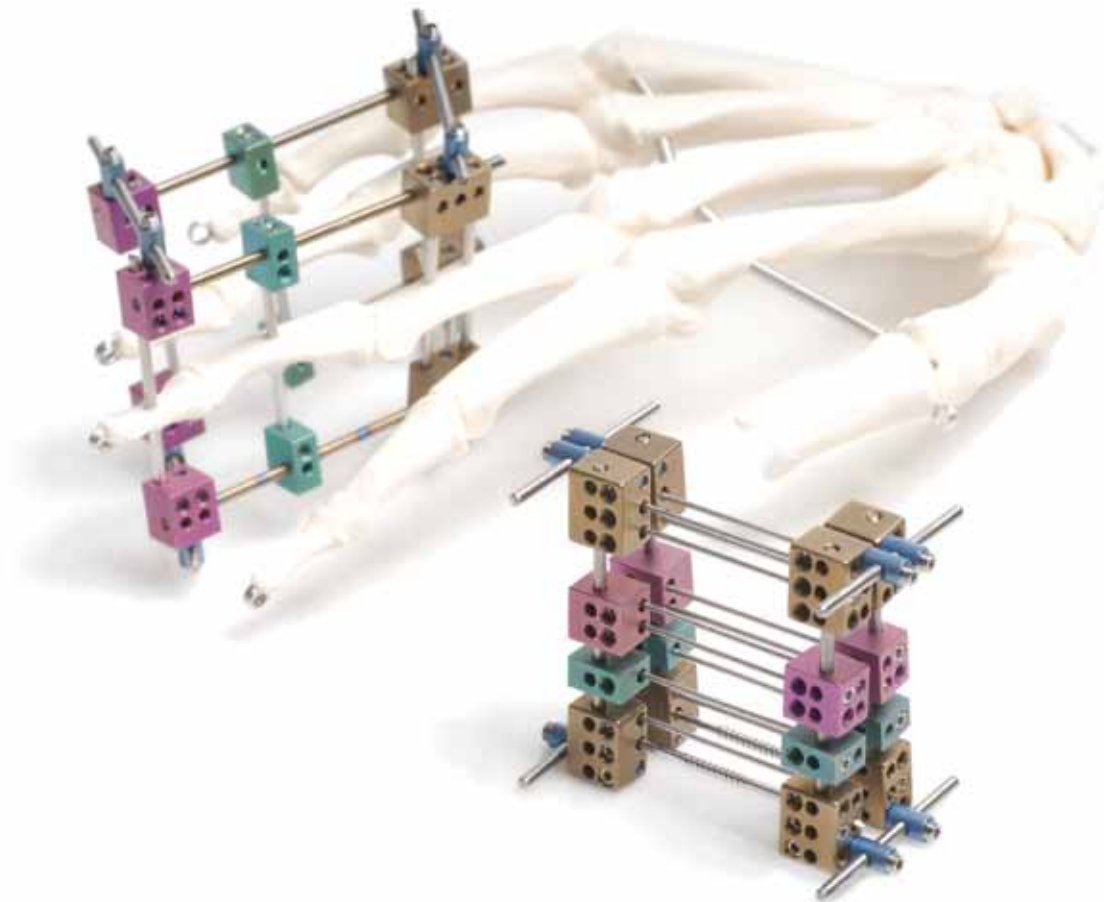
modular fixation device **Cube-Fix**

*for soft tissue and bone distraction treatment
in infants and toddlers*

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- *modular construction*
- *adaptable to smallest infant structures*
- *minimal weight*
- *removable distraction unit*

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Scopes of application for the modular fixation device **Cube-Fix**

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Introduction

The cube fix system is a very light modular mini fixing device with a removable distraction unit which can be individually adapted to the small anatomical structures of children's hands, permitting the option of using distraction procedures even in very small children, particularly also for correcting congenital deformities. The main fields of application are cross-wise soft tissue distractions to improve the options in correcting complex syndactyly and for other callus distractions to lengthen phalangeal and metacarpal bones.

The use of the fixing device has proven itself in more than 100 cases for the treatment of congenital deformities.

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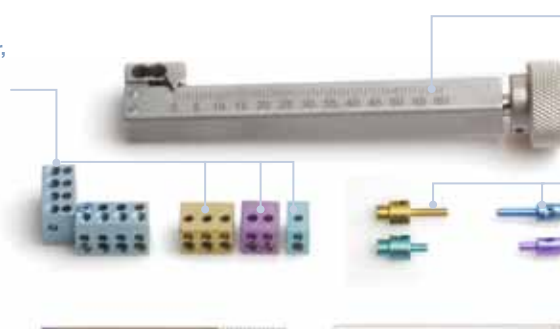
Steps of application for the modular fixation device Cube-Fix

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Application steps

In fig. 2, the starting position before distraction is shown. To perform soft tissue distraction, pins or wires are placed in the extremities. These are then fixed in the pin clamp blocks with M3 threaded pins. Now one connection pin is placed in each clamp block to connect the two blocks with a connection pin. The individual components are fastened to each other with M3 threaded pins throughout. Now the distractor can be applied for distraction.

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
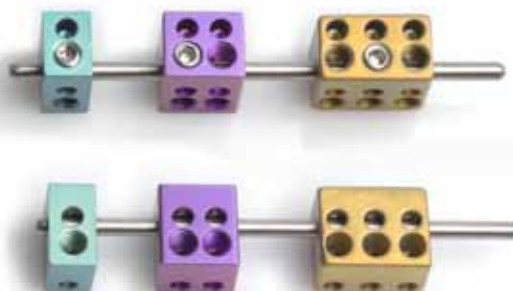
fig. 2
pin clamp block with connection pin through clamp and connection pin according to anatomical situation

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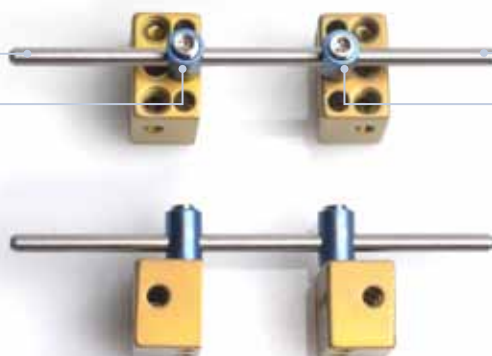connection pin Ø 2mm
clamp and connection pin

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fig. 3
starting position before distraction in lateral view

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Steps of application for the modular fixation device **Cube-Fix**

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The distractor is placed on the connection pins. The drill holes of the outriggers on the distractor fit precisely onto the connection pins.

Once the distractor has been applied, it holds itself and does not need to be held by hand.

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fig. 4
placing the distractor on the connection pin and releasing the clamp screw in the connection pin

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The through holes make it possible to release the clamp screw when the distractor is applied in order to start the distraction process in the direction of the arrow.

One turn corresponds to 0.5mm.

To monitor the covered distraction distance, there is a scale on the distraction housing.

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fig. 5
start of distraction by turning the distractor clockwise and fixation of the threaded pin

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The distractor is placed on the connection pins. The drill holes of the outriggers on the distractor fit precisely onto the connection pins. Once the distractor has been applied, it holds itself and does not need to be held by hand.

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After the distraction process, the connection pin is secured with the M3 screw in the clamp and connection pin. Then the distractor can be removed.

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fig. 6
*End of the distraction process
before taking off the distractor*

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Rev. Nr. 3 Cube 05.11 copyright © **litos/** 2011