AC-Hakenplatte & Claviculaplatten

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AC hook plate & clavicle plates

multidirectional locking screw tifix® implants
 suitable for minimally invasive procedure

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tifix® AC *product features*
hookplate

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Indication

acute acromioclavicular luxation, Tossy III,
older AC-luxation with complaints

Objective

Temporary fixation of AC luxation,
stabilisation of the fracture by means
of a bridging osteosynthesis,
functional follow-up treatment

Technical solution

Anatomically formed titanium hook plate,
two to five hole, application with multidirec-
tional fixed-angle **tifix®** screws or conven-
tional screws, reinforced hook zone.

Special feature

The end of the hook is to be placed into a
hole in the acromion, drilled by using the
template. The joint is hence stabilized whilst
maintaining movability. The hole at the end of
the hook enables the use of a guide thread,
thus allowing minimally invasive surgery.

OP-Set **tifix® AC** Hakenplatte
Art. Nr. LS-CLAV-H

surgical set **tifix® AC** hook plate
part ref. No. LS-CLAV-H

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multidirectional screw locking with tifix® screws mini 1 (blue) or non-fixed-angle with mini 1 conventional (yellow) in all plate holes possible.

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The hook can be bent in accordance with the AC incline

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
Länge / length 89 mm

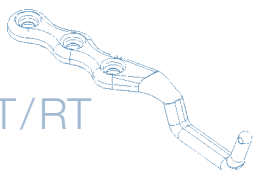
CLAVH3-FIXVLT / RT

tifix®- AC hook plates are available in different lengths with two to five holes, in a right and a left version. 2- to 4- hole plates are part of the surgical set, the 5-hole plate is optionally available

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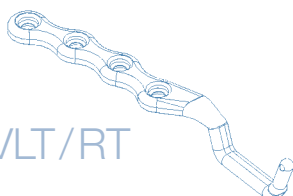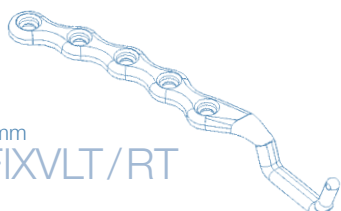
*All tifix® AC hook plates are now **reinforced** in the hook zone.*

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indication

- fracture of the lateral Clavicle
- malpositions with retained acromioclavicular Clavicle joint

goal

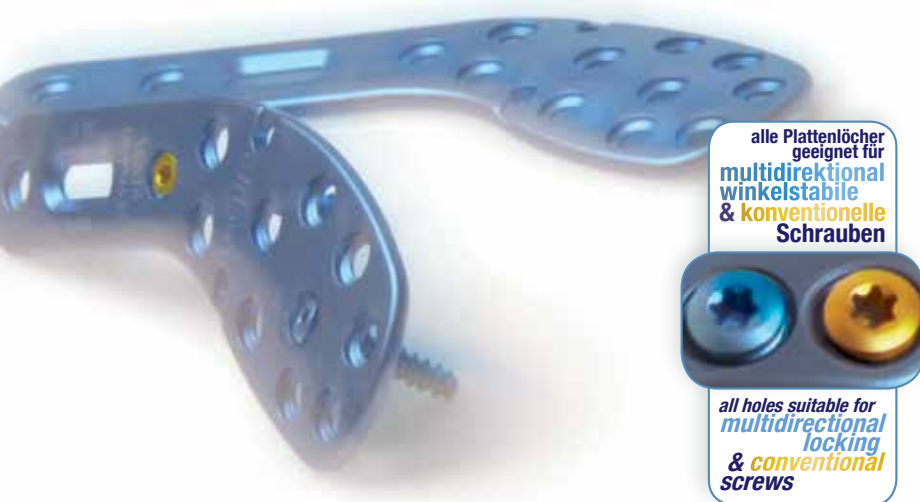
stabilization of the fracture or osteotomy with multidirectional locking screw technology implant

technical solution

anatomically shaped plate, can be used with **tifix®** mini 3 screws Ø 2.6 (blue) or mini 3 conventional Ø 2.6 (yellow)

Clin. consultation and design assistance

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tifix® Clavicula universal Produkteigenschaften

tifix® universal clavicle product features

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reduced contact area
between bone surface and
rear side of plate

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all holes suitable for
multidirectional
locking
& conventional
screws

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indication

dislocated fractures, pseudarthrosis,
fractures with vessel/nerv damage, a.o.

contraindication

infected, necrotic skin areas, open fractures
with contaminated wounds, a.o.

goal

exercise stable fixation, anatomical recon-
struction

technical solution

anatomically shaped plate with multidirec-
tional locking screw technology, can be used
with **tifix®** mini 1 (Ø 4.0) or mini 2 screws (Ø
3.5) and conventional mini 1 (Ø 4.0) or mini 2
screws (Ø 3.5), on the plate ends **tifix®**
mini 3 (Ø 2.6) or mini 4 (Ø 2.0) are matching.

Clin. consultation and design assistance
D. Wolter, Hamburg

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
tifix® universal clavicle plates are anatomically pre-shaped, and can be bent further intraoperatively. Deformation of holes while bending, will be reduced by inserting screwheads. The top side is rounded, the rear contact surface to the bone is minimized.

Short plates for the distal and proximal position as well as medium size and long plates for the medial position are available.

Bezeichnung Name	Lage position	Seite side	Loch hole	Länge length	Art.Nr. ref.no.
kurze Platte short plate	distal distal	links left	4 4	80 mm 80 mm	60208004LT 60208004LT
kurze Platte short plate	distal distal	rechts right	4 4	80 mm 80 mm	60208004RT 60208004RT
kurze Platte short plate	proximal proximal	links left	4 4	80 mm 80 mm	60108004LT 60108004LT
kurze Platte short plate	proximal proximal	rechts right	4 4	80 mm 80 mm	60108004RT 60108004RT
mittlere Platte medium plate	medial medial	links left	6 6	95 mm 95 mm	60409506LT 60409506LT
mittlere Platte medium plate	medial medial	rechts right	6 6	95 mm 95 mm	60409506RT 60409506RT
lange Platte long plate		links left	8 8	155 mm 155 mm	60315508LT 60315508LT
lange Platte long plate		rechts right	8 8	155 mm 155 mm	60315508RT 60315508RT

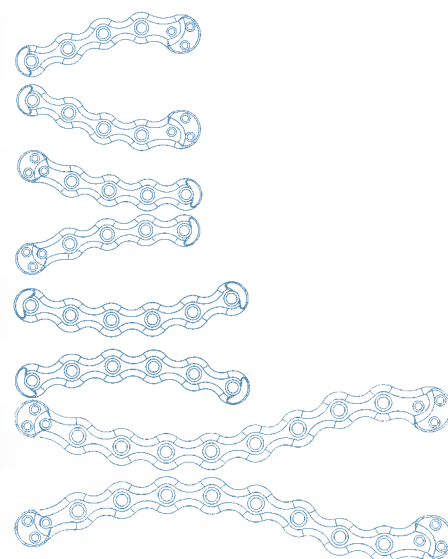Produktübersicht tifix® Claviculaplatten lateral und universal overview components tifix® clavicle plates lateral and universal

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